
Automatic Veterinary Syringes

ThaMa 313

Fixed Dosage with Clear Barrel and Vial Holder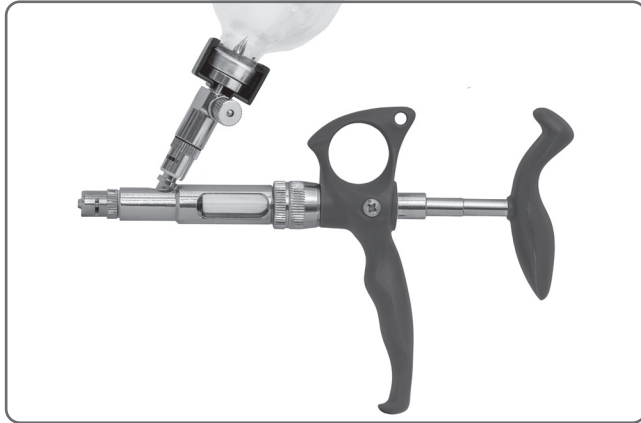

Operating Instructions:

ThaMa 313 syringe is the most versatile automatic syringe for the treatment of swine and other animals. With metal body and clear barrel.

Fitted with a unique multi-purpose connector, to administer vaccines directly from the manufacturer's vial through an easy use vial holder, or from a plastic feeding bottle. All the components are made of the finest material, oil proof, non-corrosive.

With 3 pistons of various sizes: 0.5cc, 1.0cc, 2.0cc.

Before Use:

1. It is recommended to sterilize the syringe before each use.
2. Make sure that the valve springs and washers are properly positioned into their base.
3. Check that all screwed parts are tightened.
4. Tight the body screw nut (40) to the body cover.

To set the dose:

1. In the box find the key-pin.
2. Open the body-cover (13) by gripping it with your fingers and turn.
3. Press the piston - forward to its utmost position and insert the key-pin into the hole on the piston.
4. Grip the piston and unscrew it and change it to the one you need. place the o-ring on the piston.
5. Tight the new piston gently with the key-pin.
6. Take out the key-pin from the piston.
7. Put a drop of castor-oil on the piston's O-ring.
8. Close the body-cover and tighten it.

Working with the vial holder:

1. Remove the plastic clip (42).
2. Insert the vial (43) to the vaccine bottle and push it down.
3. Return the plastic clip back – connect the vial holder to the syringe.

Pay attention that the plastic clip is properly set!

Recommended:

Use the original box for each syringe to keep it useful for long time.

All metal parts can be sterilized up to 125°C.


All other parts can be sterilized by boiling water up to 100°C.

Care and Maintenance after Use:

1. Dismantle the vial from the vial holder.
2. Pump the syringe in clear water, 6-10 times, to remove any residue of the material from the body, needles, tube and valves.
3. Open the body cover (13) and wash all parts in clear water.
4. Open the luer lock needle adapter (02) and the tubing valve housing (03) - wash them in clear water.
5. Oil the plastic cylinder after cleaning.
6. Assemble the parts together.

Note: Pay attention and take good care not to lose the small parts, like valves, and springs.

Note: It is highly recommended to oil the plastic cylinder.

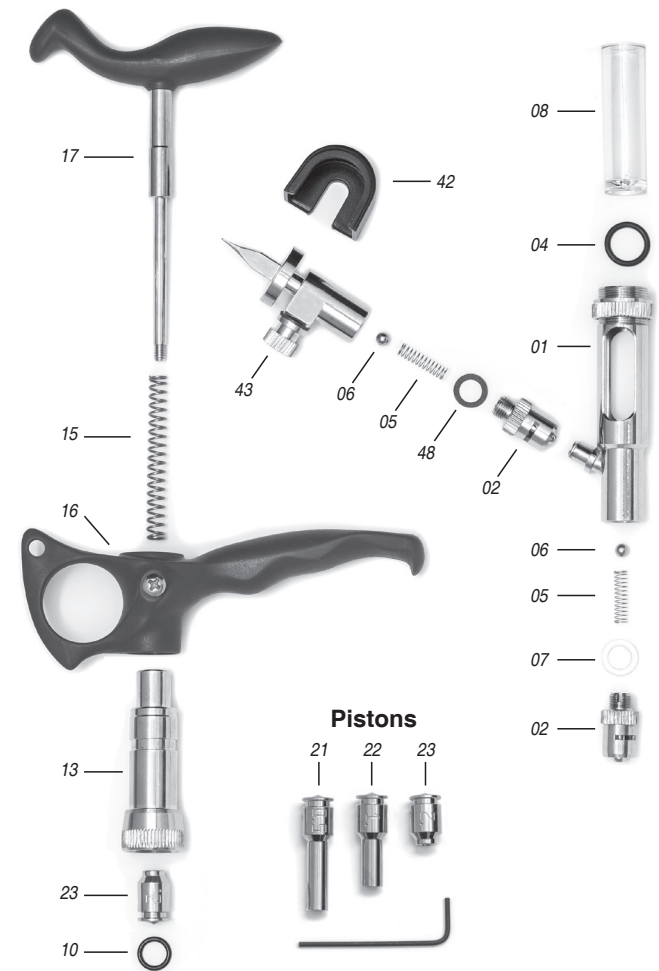- 313-01 Syringe Body
- 313-02 Luer lock, needle adapter
- 313-04 Cylinder Washer
- 313-05 Valve Spring
- 313-06 Valve stainless ball
- 313-07 Valve Plastic Washer
- 313-08 Clear Cylinder
- 313-10 Piston O-Ring
- 313-13 Body Cover
- 313-15 Handle Spring

- 313-16 Finger grip - handle
- 313-17 Upper Handle
- 313-42 Plastic Clip
- 313-43 Vial Holder - Body
- 313-48 Vial holder - Fiber washer
- 313-99 Spare Parts-Kit
- Pistons:**
- 313-21 Piston 0.5cc
- 313-22 Piston 1.0cc
- 313-23 Piston 2.0cc

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