Automatic Veterinary Syringes

ThaMa 312 Fixed Dosage with Clear Barrel 2.0cc

Operating Instructions:

This Automatic Syringe ThaMa 312 is a precision, reliable and accurate Fixed-Dosage Clear Barrel instrument, designed for mass treatment of swine.

This syringe is for use with feeding tube, that allows the injection of any kind of product for swine and other animal. All the components are made of the finest material, oils proof, non-corrosive.

With 3 pistons of various sizes: 0.5cc, 1.0cc, 2.0cc.

Before Use:

1. It is recommended to sterilize the syringe before each use.
2. Make sure that the valve springs and washers are properly positioned into their base.
3. Check that all screwed parts are tightened.
4. Tight the body screw nut (40) to the body cover.

To set the dose:

1. In the box find the key-pin.
2. Open the body-cover (13) by gripping it with your fingers and turn.
3. Press the piston - forward to its utmost position and insert the key-pin into the hole on the piston.
4. Grip the piston and unscrew it and change it to the one you need. place the o-ring on the piston.
5. Tight the new piston gently with the key-pin.
6. Take out the key-pin from the piston.
7. Put a drop of castor-oil on the piston's O-ring.
8. Close the body-cover and tighten it.

Preparing the vial for vaccination:

1. Insert the long canula through the rubber cover; be sure to insert it down to the bottom of the vial.
2. Connect the plastic tube to the long canula, on one end, and to the syringe inlet on the other end.
3. Operate the syringe handle until the vaccine is being pumped from the vial up to the syringe.

Recommended:

Use the original box for each syringe to keep it useful for long time.

All metal parts can be sterilized up to 125°C.


All other parts can be sterilized by boiling water up to 100°C.

Care and Maintenance after Use:

1. Pump the syringe in clear water, 6-10 times, to remove any residue of the material from the body, needles, tube and valves.
2. Open the body cover (13) and wash all parts in clear water.
3. Open the luer lock needle adapter (02) and the tubing valve housing (03) - wash them in clear water.
4. Oil the plastic cylinder after cleaning.
5. Assemble the parts together.

Note: Pay attention and take good care not to lose the small parts, like valves, and springs.

Note: It is highly recommended to oil the plastic cylinder.

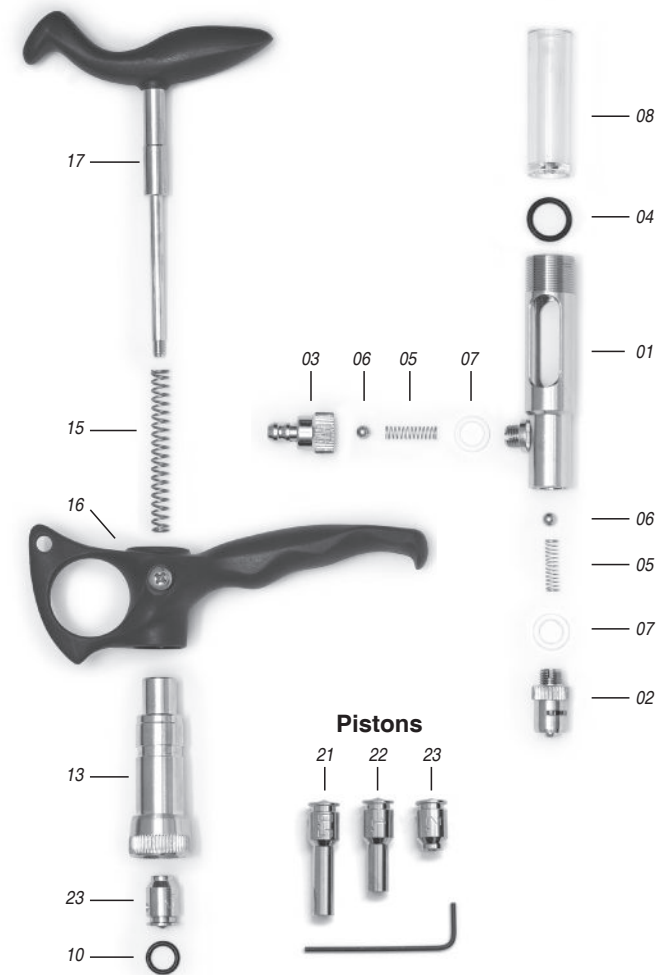- 313-01 Syringe Body
- 313-02 Luer lock, needle adapter
- 312-03 Tubing valve housing
- 313-04 Cylinder Washer
- 313-05 Valve Spring
- 313-06 Valve stainless ball
- 313-07 Valve Plastic Washer
- 313-08 Clear Cylinder
- 313-10 Piston O-Ring

- 313-13 Body Cover
- 313-15 Handle Spring
- 313-16 Finger grip - handle
- 313-17 Upper Handle
- 313-99 Spare Parts-Kit
- Pistons:**
- 313-21 Piston 0.5cc
- 313-22 Piston 1.0cc
- 313-23 Piston 2.0cc

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