


## How to lock the bridge to the ThaMa 422 Syringe


Step 1 –  
Place the in front of the syringe

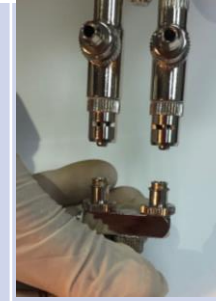


Step 2 –  
Attached the bridge to the syringe



Step 3 –  
Lock the bridge by twisting the wheels to the right.  
Do it by **hand only**.  
Do **NOT** use pliers.

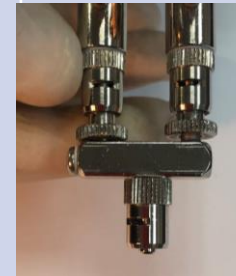

Step 4 –  
After locking - the wheels maybe loose.  
This is OK

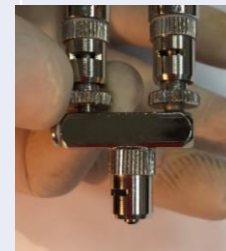

## How to unlock the bridge


### Step 1 –

Twist the wheel to the left by hand.  
Try to release the bridge from the syringe



### Step 2 –

**Only** if needed use pliers.  
Gently twist the wheels to release the bridge.

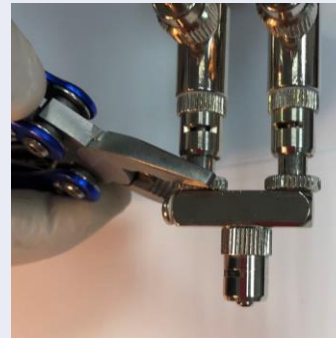

See here a link to a full tutorial movie:

[ThaMa 422 - How to lock the bridge](#)



For any question, please contact us at: [info@thama-vet.com](mailto:info@thama-vet.com)