
Automatic Veterinary Syringes

ThaMa 422 Automatic Double Syringe

2 different dosages of various vaccines
at once with different pistons
0.05cc & 0.1cc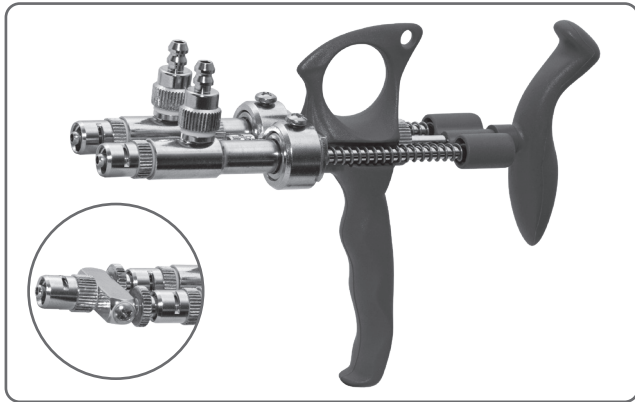

Instructions:

This ThaMa 422 is a precise, reliable and accurate double syringe. ThaMa 422 is used for vaccinate two different dosages of various vaccines at once. Each syringe comes with a set of pistons.

Before Use:

1. It's recommended to sterilize the syringe before each use.
2. Check that all screwed parts are tightened.
3. Make sure that the valve springs and washers are properly positioned into the base.

How to set the dosage:

1. Unscrew the banding screw (#26) and pull out the syringe.
2. Unscrew the body cover (#13) and pull out from the body (#1).
3. Insert the key-pin (#20) into the hole on the piston.
4. Hold the top nut with tool and unscrew it out.
5. Take the spring (#15) and the body cover (#13) and put them on the new piston (#11) with the measure you need.
6. Replace the cover body, spring and the screw nut M4 (#12) in order.
7. Tight the screw nut (#12): insert the key-pin into the hole on the piston and tight gently with tool.
8. Put the syringe back in its housing (#25) (make sure that the top nut is in its hole at the piston shove) and screw the banding screw (#26) tightly.

Make sure that you set the dosage in both syringe.

**One of the syringes is marked!
That makes it easier to remember in which
syringe you have one of different dosages
during the working day.**

How to set the Luer lock 2-to-1 bridge (#41):

1. Attach the luer lock bridge by inserting it into the syringe.
2. Secure the bridge by twisting the rings of the bridge.


Care and Maintenance after Use:

1. Dismount the needles from the syringes.
2. Take out the syringes from the syringe housing (unscrew #26).
3. Open the body cover and wash it, open the luer lock needle adapter and the tubing valve housing – wash them in clear water.
4. Before using again, verify that all the parts are in their place and put a drop of castor oil (supplied) on the O-ring.

Do not open the upper handle for cleaning!

Sterilization:

All parts of the syringe can be sterilized. It is recommended to sterilize after every use. Up to 120°C in autoclave.

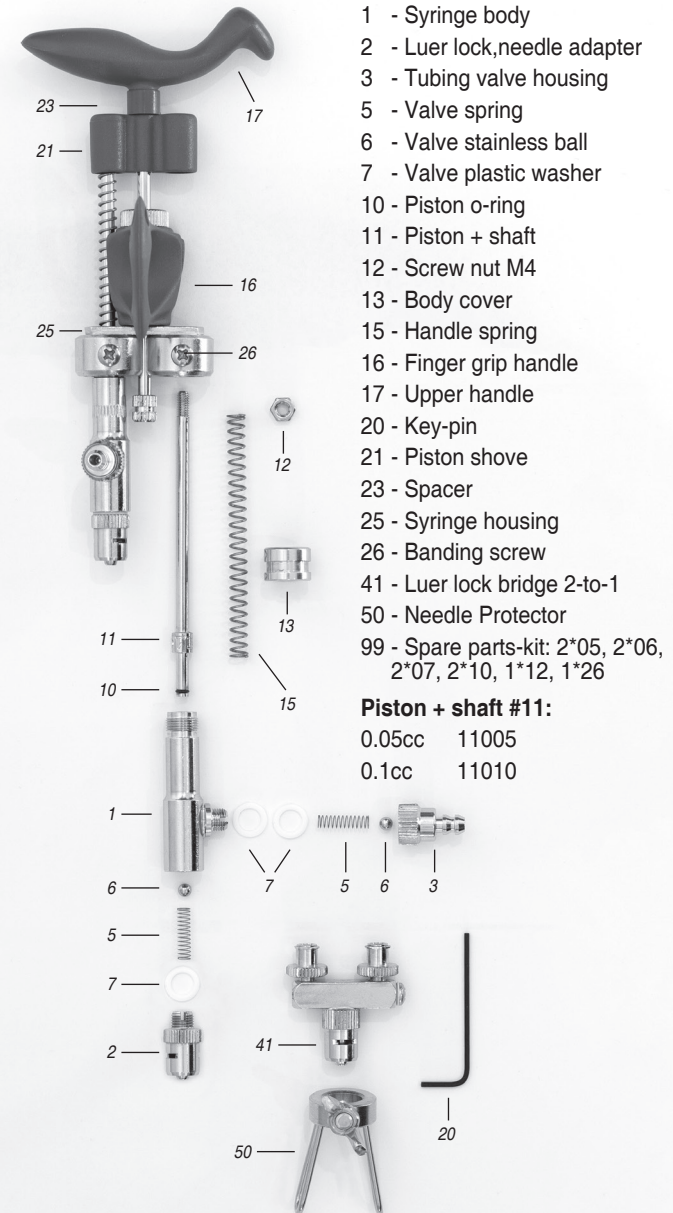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