CleanMark Instructions manual

English

Have all the parts handy including your ThaMa Syringe, CleanMark kit (CleanMark, 3 Pads, 2 Adjusting pins, 1gr Food color)

Attach the CleanMark **1**

- Position the CleanMark onto the ThaMa syringe.
- Secure the CleanMark gently using the butterfly screw to ensure proper attachment.

Attach the Needle **2**

- Install the needle on the syringe.
- The CleanMark is pre-adjusted for a 1 ½" needle. For other needle lengths, adjust the CleanMark length using the included pins.

Install the Pad **3**

- Slide the pad into the CleanMark.
- Ensure it is properly seated to

prevent it from moving during use.

Prepare the Colored Cleaning Solution **4**

It is recommended to wear gloves.

- Use the provided CleanMark plastic container, fill it with 10ml of alcohol, and add food coloring. You can use any available food color in the market, liquid or powder.
- Mix the solution well.

Prime the Pad **5**

- Soak the pad in the colored alcohol solution until fully saturated.

Start Vaccination

The ThaMa syringe with CleanMark is now ready for use. Operate the syringe as usual.

Important Notes

- **Initial Use:** Some color dripping may occur during the first few injections. This will stabilize after 2-3 uses.
- **Handling the Cleaning Solution:** The colored alcohol can stain surfaces and skin. It is recommended to wear gloves.
- Food coloring stains can be removed with soapy water.

Maintenance and Storage

- After use, disassemble the CleanMark accessory and clean all components with soapy water to remove alcohol and color residues.
- Dry thoroughly before storing to maintain product longevity.

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